

Liofilchem®

ComASP™ Colistin

0.25-16 µg/mL

Compact Antimicrobial Susceptibility Panel containing colistin (polymyxin E) in 7 two-fold dilutions, that allows for testing up to four samples per panel in accordance with international standards (i.e. EUCAST, CLSI, ISO 20776-1:2006).



Packaging 16 tests
Catalog Ref. no. 75001

Trademark notice


ComASP™ is a trademark under registration process by Liofilchem®. The product represented in the photographs of this leaflet show the SensiTest panels before their rename into the current ComASP™ trademark. Photographs of the panels with ComASP™ printout were not available at the moment of printing the present leaflet. The SensiTest trademark is no longer used by Liofilchem®.



Liofilchem® srl

Zona industriale, 64026 Roseto degli Abruzzi (Te) Italy
www.liofilchem.com


ref. 6552490
Rev. 1 - 18.04.2018
© Liofilchem 2018
printed in Italy

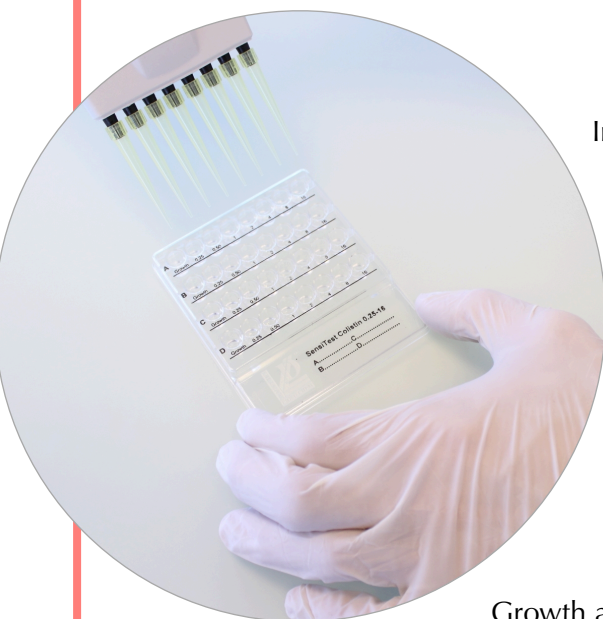

Incubation at 37°C for 16-20 hours in ambient air.

At the end of the incubation period watch for the growth in the wells and determine the MIC, i.e. the lowest concentration of antibiotic that inhibits visible growth.

Growth appears as turbidity or as a button at the bottom of the well (compare with the amount of growth in the growth-control well).

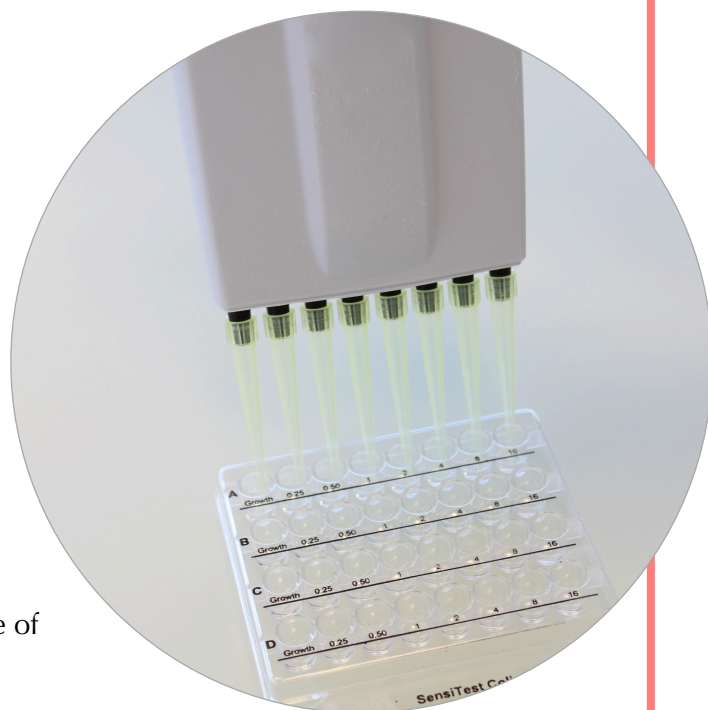

The result can be read visually or automatically. Reading by the naked eye can be improved by use of bright indirect lighting against a dark background.



Liofilchem® and the Liofilchem company logo are registered trademarks of LIOFILCHEM s.r.l.



Liofilchem® srl

Zona industriale, 64026 Roseto degli Abruzzi (Te) Italy
www.liofilchem.com

ref. 6552490
Rev. 1 - 18.04.2018
© Liofilchem 2018
printed in Italy